

LACTO[®]

EYE DROPS

Lubricant Ophthalmic Solution
with Hypromellose

NOVAX[®]
PHARMA



Composition:

Lactoferrin, Hydroxypropyl Methylcellulose, Sodium N-Hydroxymethylglycinate, EDTA Disodium salt, Potassium Hydroxide, Monobasic Sodium Phosphate, Dibasic Sodium Phosphate, Sodium Chloride, purified water.

Description:

LACTO[®] is an ophthalmic solution containing hypromellose which lubricates and hydrates the ocular surface.

LACTO[®] is useful in all cases of tear film alteration, caused by: exposure to sunlight, atmospheric agents, exposure to video terminals.

The preservative system includes lactoferrin, a substance normally present in the tear fluid, which, seizing the iron, helps to inhibit microbial growth.

Indications:

LACTO[®] is indicated in the treatment of dry eye and in the treatment of symptoms related to inflammatory states in progress.

Instruction for use:

The duration of treatment and the dosage should be indicated by your eye specialist. The recommended dosage is 1-2 drops in the affected eye.

1. Wash your hands.
2. Unscrew the cap.
3. Hold the bottle upside down between the thumb and the other fingers.
4. Pull down the lower eyelid.
5. Put the tip of the bottle near the eye without touching the eye or adjacent regions.
6. Apply the recommended number of drops and close the eyelids gently to distribute the product on the ocular surface evenly.
8. Repeat the procedure for the other eye if indicated by your eye specialist.
9. Close the bottle tightly after application, to avoid contamination.

Warnings:

- Cases of corneal calcification associated with the use of eye drops containing phosphates have been reported very rarely in patients with significantly damaged corneas. In case of band keratopathy or calcium deposits on the cornea, consult your specialist before using this product;

- The product is sterile until opening;
- To prevent possible contamination, do not let the tip of the dropper come into contact with the eye or any surface;
- Close the bottle tightly after use;
- Do not use in case of known hypersensitivity to any of the ingredients of the product;
- Do not use the product if the package is not intact;
- In case of adverse reactions, discontinue immediately the treatment and consult your physician;
- Do not swallow;
- Do not use the product after the expiration date shown on the pack;
- The product must be used within 30 days after opening. Discard the remaining product;
- In case of persisting problems, consult your physician;
- Do not use the product continuously for more than 30 days.

Storage:

- Protect from heat sources and keep out of reach of children;
- Store below 30°C.



3 ml (Free Sample)

15 ml

License: NOVAX[®] PHARMA

Le Coronado - 20, Av. de Fontvieille
MC 98000 MONACO
www.novaxpharma.com

CE
1370

V1 - Rev. 04/2019