



LUXCLEAR®

PRESENTATION

Bottle of 100 ml or 360 ml - STERILE

COMPOSITION

Pluronic, disodium edetate and PHMB, isotonic solution buffered.



AVAILABLE
IN 360 ML
& 100 ML

PRESERVATIVE

PHMB

DIRECTION FOR USE

- ▶ Wash and then dry your hands thoroughly.
- ▶ Fill both sections of the lens case with the solution, making sure that it is empty and clean first, and place each contact lens in the appropriate compartment of the lens case.
- ▶ Close the lens case firmly and leave the contact lenses immersed in the solution for at least 4 hours.
- ▶ Shake the lens case gently before opening.

- ▶ Wash and dry your hands carefully before removing the left contact lens from the left compartment of the lens case.
- ▶ With fresh solution, gently rub the contact lens on both sides in the palm of the hand.
- ▶ Rinse the contact lens with fresh solution before applying it to the left eye.
- ▶ Repeat the same steps for the right lens before applying it to the right eye.
- ▶ Empty the lens case after each use.

BENEFITS

- ▶ **Hydrates, disinfects, cleans, removes proteins, lubricates and rinses.**
- ▶ **When used daily, it will prevent the deposits of proteins which can damage your lenses and may lead to eye infections and reduction of the lens life and wearing time.**

SAFE TO USE WITH CONTACT LENSES

Yes

SHELF LIFE

36 months

PERIOD AFTER OPENING

90 days

CLASS

IIB