



NAVIBLEF® DAILY CARE

PRESENTATION

Bottle of 50 ml foam

COMPOSITION

Melaleuca Alternifolia Oil (Tea Tree Oil), Anthemis Nobilis Oil (Chamomile Oil), D-Panthenol, Allantoin, Taurine and purified Water.

PRESERVATIVE

Microglicin-50, Boric Acid.

DIRECTION FOR USE

Close the eye and massage the eyelid and eyelashes with **NAVIBLEF® DAILY CARE**. Do not touch or open the eyes during the application.

Leave the foam for approximately 60 to 80 seconds.

After each application rinse the eyelashes and eyelids with warm water.

Note: Repeat the same steps for the other eye.

BENEFITS

- ▶ Effectively removes excessive oil, pollen, and other debris from the eyelids which increase the risk of irritation, blepharitis due to the presence of Demodex (eyelid mites) and bacterial infestations or dry eye symptoms.
- ▶ Recommended to be used as maintenance care, for patients that have followed the blepharitis treatment with **NAVIBLEF® INTENSIVE CARE**.

SAFE TO USE WITH CONTACT LENSES

No

SHELF LIFE

36 months

PERIOD AFTER OPENING

6 months

CLASS

I