



NAVIBLEF® WIPES

PRESENTATION

20 Wipes - STERILE

COMPOSITION

Terpinen-4-ol, Sodium Hyaluronate and Aloe Vera.

PRESERVATIVE

Phenethyl Alcohol, Ethylhexylglycerin

DIRECTION FOR USE

CLOSE YOUR EYES DURING APPLICATION, LET THE SOLUTION DRY UP BEFORE OPENING THE EYES AGAIN (1 mn).

- Wash, rinse and dry hands and face.
- Tear open the aluminum sachet remove and unfold the wipe.
- Close your eyes tightly and gently wipe the lids using side to side motion for approximately 30 seconds or dab the wipes on the affected area. The solution helps to inhibit the growth of the pathogens and not the scrubbing action. The scrubbing in certain cases may cause patient discomfort.

- Keep the eyes closed for 1 minute to allow to dry the treated surface.
- Do not wash off the dried formulation, let it act for a while. For optimum results do not rinse immediately but wait 5 to 10 minutes before rinsing with running water.
- Once finished cleaning the first eyelid, turn the wipe over and use the opposite side to repeat the procedure with your other eyelid. You can use one wipe for both eyelids unless otherwise directed by your eye specialist.
- Discard wipe after use.

BENEFITS

- ▶ **A formulation recommended for daily hygiene and removal of ocular secretions and debris of eyelids and eyelashes caused by allergic and inflammatory conditions associated with Demodex, Blepharitis, Blefaro-Conjunctivitis, Meibomian gland dysfunction (MGD), Rosacea, burning, itchiness, irritation, redness swelling and other eyelid margin diseases. Also recommended for the use after post ocular and cataract surgery.**
- ▶ **In case of abundant secretions and residues, daily application is recommended.**

SAFE TO USE WITH CONTACT LENSES

No

SHELF LIFE

30 months

PERIOD AFTER OPENING

N/A

CLASS

Is