navilipo

DRY EYES

EYE DROPS

Lubricating and restoring liposomal eye drops with Sodium Hyaluronate & Vitamin E

EN COMPOSITION:

Solution with Liposomes, Aloe Vera gel 1%, Chamomile Extract, Sodium Hyaluronate, Vitamin E-TP, Disodium Edetate, PHMB in an Isotonic buffered solution.

PS: Does not contain Chlorhexidine, Thimerosal, or Benzalkonium Chloride.

DESCRIPTION:

NAVI[®]LIPO is a lubricating and restoring ophthalmic liposomal solution made up from phospholipids which are natural constituents of the lachrymal fluid which helps restore the lipid layer. The sub-microscopic lipid particles dispersed in the micro-emulsion protect the ocular tissue, by forming a thin hydrophobic barrier, which mimics the lipid layer naturally found in the tear film.

INDICATIONS:

NAVI[®]LIPO eye drops helps restoring the ocular surface micro-environment altered by the increased tendency to evaporate, instability of tear film, traumas, ocular surgery, wear of contact lenses, prolonged exposure to computer terminals.

DIRECTION FOR USE:

- 1 - Shake before use.
- 2 - Wash your hands before using the eye drops bottle.
- 3 - Instill 1 or 2 drops of the solution in each eye, as many times as needed or according to your specialist's recommendation.

NAVI[®]LIPO can be used with all types of contact lenses.

WARNINGS:

- In case of eye irritation, stop using the product and contact a physician.
- Do not use if you are allergic to any of the product components.
- Do not touch the eyes with the tip of the bottle.
- Keep out of the reach of children.
- Do not ingest.

- Do not use the product after the expiry date.
- Use within 28 days of first opening.
- Sterile until first opening.
- Close the bottle tightly immediately after use.

CONTRAINDICATIONS:

- None known.

STORAGE:

- Keep away from heat sources.
- Store in a dry place, between 5 and 35°C.

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