EN *For external use only*
COMPOSITION

Terpinen-4-ol, Anthemis Nobilis (Chamomile) Essential Oil, D-Panthenol, Allantoin and purified Water.

INDICATIONS

NAVIBLEF[®] TERP INTENSIVE CARE has an improved formula which induces an efficient mechanical removal of severe ocular secretions and debris from eyelashes and eyelids.

These are caused by inflammatory conditions such as dryness, allergies, blepharitis and blepharconjunctivitis, resulting from the presence of Demodex (eyelid mites) and bacterial infestations, dry eyes, contact lens wear...

This foam can also be used to clean the base of the eyelashes and eyelid margins before and after surgery.

DESCRIPTION

NAVIBLEF[®] TERP INTENSIVE CARE contains a high concentration of Terpinen-4-ol, one of the components of tea tree oil, which thanks to its oily texture, helps to eliminate the most difficult secretions of eyelids and eyelashes.

NAVIBLEF[®] TERP INTENSIVE CARE contains Allantoin to help keep the skin hydrated.

WHAT IS BLEPHARITIS?

It is a chronic inflammation of the eyelids and is one of the most common disorders of the eye usually due to the presence of Demodex (eyelid mites) and bacterial infestations. This is often the main cause of eye discomfort, redness and tearing.

Other eye symptoms of blepharitis include burning, itching, light sensitivity, irritation, sandy and gritty sensation that is more accentuated upon waking.

Blepharitis is a chronic disease for which there is no cure and requires a long-term treatment, hygiene and maintenance to keep it under control and prevent subsequent related complications linked to this type of conditions.

PACKAGING

50 ml

REF

5 ml (free sample)

REF

PROPERTIES

• NAVIBLEF[®] TERP INTENSIVE CARE is a non-greasy and non-irritant formulation, for topical use.

• NOT TESTED ON ANIMALS.

INSTRUCTIONS FOR USE

1 - Before the application of the foam, wash your hands and then clean the fingertips with NAVIBLEF[®] TERP INTENSIVE CARE.

2 - Press the delivery pump and place the foam on the clean fingertips. When used before or after surgery it is recommended to place the foam on sterile gauze.

3 - Close the eye and massage the eyelid and eyelashes with NAVIBLEF[®] TERP INTENSIVE CARE. Do not touch or open the eyes during the application. Let the product dry for 1 minute before opening the eye to avoid the product entering the eye. Once dry, you can open the eyes but keep the foam on for at least 3 to 5 minutes before rinsing.

4 - After each application rinse the eyelashes and eyelids with warm water.

Note: Repeat the same steps for the other eye.

Repeat this treatment 2 - 3 times a day for a period between 7 to 14 days, for maximum efficacy.

PRECAUTIONS

- Always wash your hands before and after use.
- Remove contact lenses before the application of the foam. Ensure all foam and residues are removed before re-inserting lenses.
- Keep your eyes closed and let the foam dry before opening the eyes.
- The product is intended for the application on the margins of eyelids and eyelashes. Do not apply directly in the eyes.
- For external use only.
- Keep out of the sight and reach of children.
- Don't use the product in children under 6 years old. For children over 6 years old the application must be performed by an adult and only after your specialist's advice.
- In case of use in pregnant or lactating women it is advised to consult your specialist beforehand.
- Use hypoallergenic eye makeup and delicate skin lotions around your eyes during use.
- Do not use in case of known hypersensitivity to one of the constituents of the product.
- If a reaction to the product occurs, stop use immediately, rinse your eyes with water and contact your eye specialist if the symptom continues.
- Discard remaining solution 3 months after first opening.
- Do not use if the bottle is damaged.
- Do not use after the expiry date.

STORAGE

• Store between 5 and 35°C and protect from a direct source of light.

NOVAX[®] PHARMA
Le Coronado
20, Av. de Fontvieille
MC 98000 MONACO
www.novaxpharma.com

CE
0051

V2 - Rev. 14/07/2020

PRESENTATIONS: NAVIBLEF[®] TERP is presented in 2 formulations:

- NAVIBLEF[®] TERP INTENSIVE CARE to eliminate severe ocular secretions and debris due to infections caused by bacteria and Demodex (red box).
- NAVIBLEF[®] TERP DAILY CARE for the maintenance care of eyelids and eyelashes (blue box).

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