

**NEW**

# Introducing **navibler®** *green*

## **OPHTHALMIC WIPES**

disposable sterile wipes for periorcular treatment  
in children and adults.

**100% cotton wipes with Green Tea,  
Hyaluronic Acid and Nettle**



**MEDICAL DEVICE**  
CLASS I



Antioxidant and refreshing,  
reducing the sensations of discomfort  
and visual fatigue.

A natural way to help keep  
your eyelids and eyelashes clean,  
comfortable and healthy.

**THIS IS NOT A COSMETIC WIPE,  
BUT A TRUE TREATMENT  
FOR YOUR EYE PROBLEMS.**

**SUITABLE FOR CONTACT LENS WEARERS.**

**INDICATED FOR TOPICAL TREATMENT OF THE  
EYELIDS AND EYELASHES FOR BOTH ADULTS  
AND CHILDREN, EVEN IN THE PRESENCE OF  
SECRETIONS RESULTING FROM :**

- ALLERGIES
- INFLAMMATION AND/OR INFECTIONS, SUCH AS BLEPHARITIS AND BLEPHAROCONJUNCTIVITIS
- MEIBOMIAN GLAND DYSFUNCTION (MGD)

**PARTICULARLY INDICATED FOR THE PRE-  
AND/OR POST-OPHTHALMIC SURGERY  
TREATMENT.**





## WHAT IS BLEPHARITIS?

The term « blepharitis » refers to various types of inflammation and infections affecting the eyelids, eyelid margins, eye lashes and meibomian glands.


Blepharitis is a common disorder throughout the world.

### Symptoms of blepharitis include :

- Itching and / or burning of the eyes
- Redness of the eyes and eyelids
- Excess tearing
- Flaking of the skin on the lids
- Foamy or frothy discharge from the eyes
- Crusting at the lid margins (particularly after waking in the morning)
- 'Foreign Body Sensation' in the eyes
- Reduced vision
- Sensitivity to light (photophobia)
- Plugged or partially plugged meibomian glands

Unfortunately, blepharitis is typically a chronic condition that has no cure, but it can be managed with long-term eyelid hygiene, medicine and allergy control.

*Bibliography: How to treat Blepharitis and Eye Allergies – 4th Edition Copyright 2017 (1st Edition Copyright 2006) – Dr Asher Neren – All Rights Reserved*

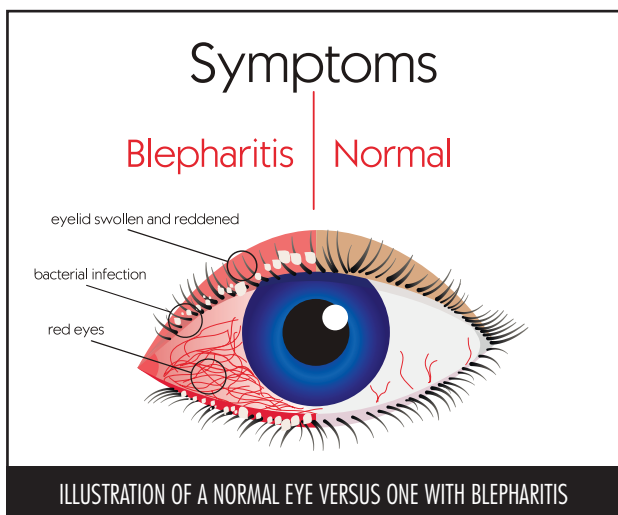

## COMPOSITION

2,8 g of an aqueous solution: Aqua, Propylene Glycol, Urtica Dioica Leaf Extract, Camellia Sinensis Leaf Water, Sodium Chloride, Phenethyl Alcohol, Sodium Diphosphate, PEG-40 Hydrogenated Castor Oil, PEG-40 Castor Oil, Polysorbate 20, Sodium Hyaluronate, Tocopherol, Ethylhexylglycerin, Sodium Phosphate, Disodium EDTA, Phenoxyethanol, Trisodium Citrate Dihydrate, Tetrasodium Glutamate Diacetate.

## INDICATIONS

**NAVIBLEF® GREEN** is indicated for topical treatment of the eyelids and eyelashes for both adults and children, even in the presence of secretions resulting from allergies, inflammation and/or infections, such as blepharitis and blepharoconjunctivitis of various kinds. Its use is particularly indicated for the pre- and/or post-ophthalmic surgery treatment.

## DIRECTIONS FOR USE

- Having cleaned your hands carefully, open the sachet and extract the wipe.
- Apply the wipe to the eyelid area, wiping it delicately to allow for adequate cleansing, removal of ocular secretions, and any other types of deposit (smog, airborne particles, dust, etc.).
- In the case of particularly persistent secretions or the occlusion of a tear duct, it is advisable to warm the closed sachet for a few minutes in lukewarm water (approx .40°C) and then use the wipe as a hot compress, massaging the area delicately.
- On the other hand, in case of phlogosis, it is advisable to cool the sachet in the refrigerator for a few hours and then apply the open wipe to the eye, keeping it in place for a few minutes as a cold compress.

## WARNINGS

- Product sterility is guaranteed only if the sachet is intact.
- The wipe is disposable. Reusing it may spread infection.
- Keep in a cool and dry place.
- Keep out of reach of children. For external use.
- Discontinue treatment in the case of evident irritation or hypersensitivity to the product. In case of infection do not use the same wipe for both eyes. Disposable.
- Read instructions carefully.

## PACKAGING



BOX OF 20 DISPOSABLE STERILE WIPES

## VITEVAX® PHARMA

Le Coronado  
20, Av. De Fontvieille  
MC 98000 MONACO  
www.vitevaxpharma.com



STERILE R



CE

0425

BR NVBWG REV 00 16/05/2022

### PROMEIA S.P.A

Corso della Vittoria 1585/1533  
I-21042 Coronno Pertusella (VA) - ITALIA

The information reported here is meant to be used only by physicians